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
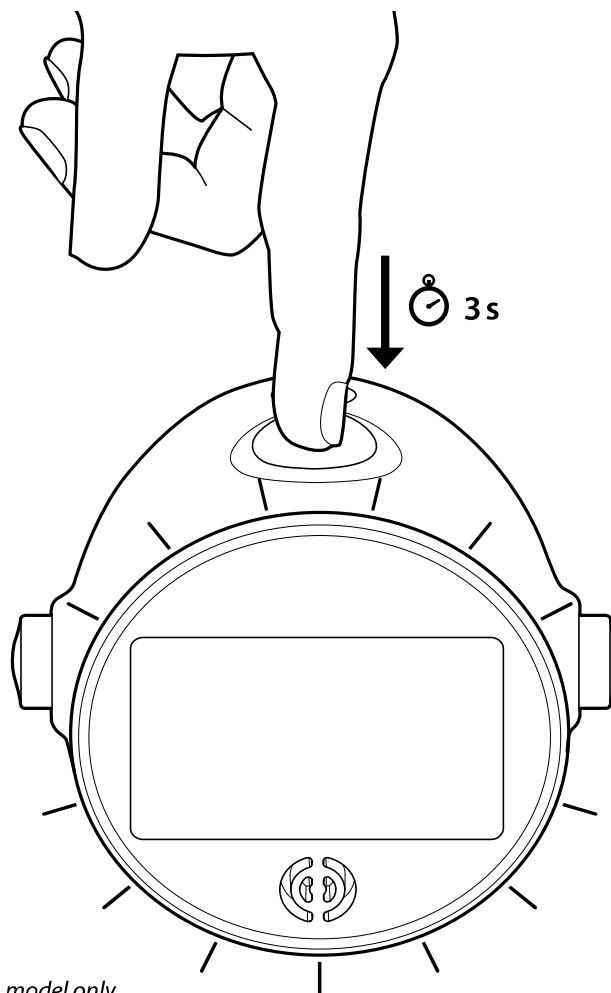
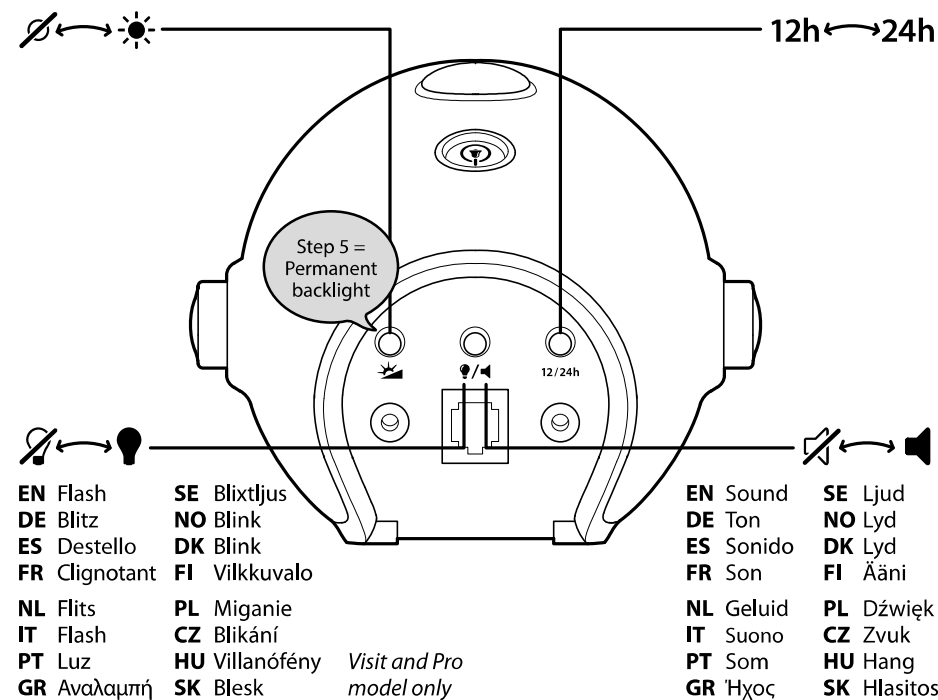
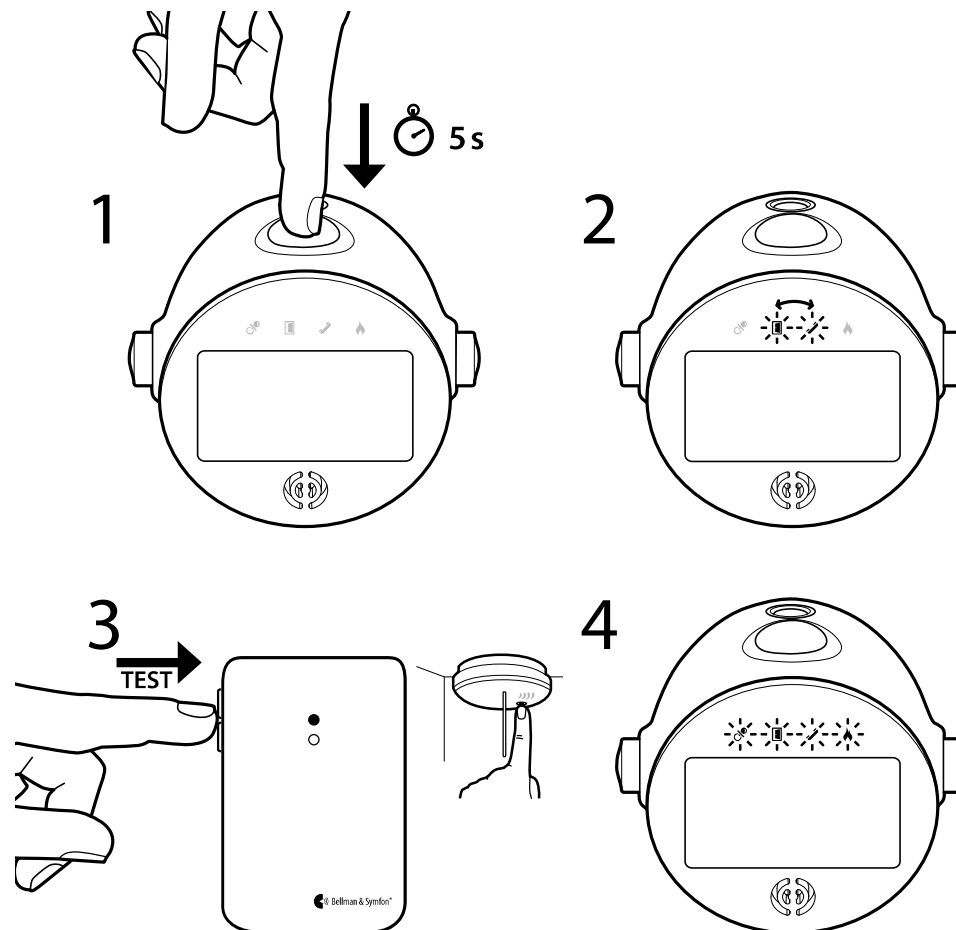
Pairing is only possible on the Visit model.

EN Time	SE Tid	EN Snooze	SE Snooze	EN Alarm	SE Alarm
DE Zeit	NO Tid	DE Schlummern	NO Slumre	DE Alarm	NO Alarm
ES Hora	DK Tid	ES Repetición	DK Slumre	ES Alarma	DK Alarm
FR Heure	FI Aika	FR Répétition	FI Torkku	FR Alarme	FI Herätys
NL Tijd	PL Czas	NL Snooze	PL Drzemka	NL Alarm	PL Alarm
IT Ora	CZ Čas	IT Snooze	CZ Odložení buzení	IT Sveglia	CZ Alarm
PT Hora	HU Idő	PT Snooze	HU Szundi	PT Alarma	HU Ébresztés
GR Ώρα	SK Čas	GR Αναβολή	SK Opakované budenie	GR Ξυπνητήρι	SK Budík

EN Flash	SE Blixtljus	EN Speaker	SE Högtalare	EN Night light	SE Nattlampa
DE LED-Blitz	NO Lys	DE Lautsprecher	NO Høytaler	DE Nachtlicht	NO Nattlys
ES Destello	DK Blink	ES Altavoz	DK Højttaler	ES Luz nocturna	DK Natlys
FR Clignotant	FI Vilku	FR Haut-parleur	FI Kaiutin	FR Veilleuse	FI Yövalo
NL Flits	PL Miganie	NL Luidspreker	PL Głośnik	NL Nachtluchtje	PL Lampka nocna
IT Flash	CZ Blikání	IT Altoparlante	CZ Reprodukto	IT luce notturna	CZ Noční osvětlení
PT Luz	HU Villanófény	PT Altifalante	HU Hangszóró	PT Luz noturna	HU Éjjeli fény
GR Αναβοσβήνει	SK Blesk	GR Ηχείο	SK Reprodukto	GR Νυχτερινό φως	SK Nočné svetlo

EN Brightness	SE Ljusstyrka	EN Flash/Sound	SE Blixtljus/Ljud	EN Format	SE Format
DE Helligkeit	NO Lysstyrke	DE Blitz/Ton	NO Lys/Lyd	DE Zeitformat	NO Format
ES Brillo	DK Lysstyrke	ES Destello/Sonido	DK Blink/Lyd	ES Formato	DK Format
FR Luminosité	FI Kirkkaus	FR Clignotant/Son	FI Vilku/Ääni	FR Format	FI Formaatti
NL Helderheid	PL Jasność	NL Flits/Geluid	PL Miganie/dźwięk	NL Indeling	PL Format
IT Luminosità	CZ Jas	IT Flash/Suono	CZ Blikání/Zvuk	IT Formato	CZ Formát
PT Luminosidade	HU Fényerő	PT Luz/Som	HU Villanófény/hang	PT Formato	HU Formátum
GR Φωτεινότητα	SK Jas	GR Αναλαμπή/Ήχο	SK Blesku/Zvuk	GR Μορφή	SK Formát

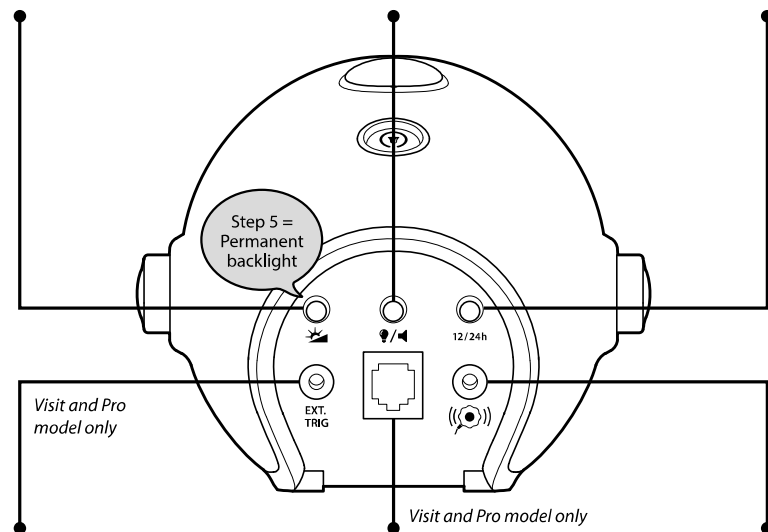EN Ext. trig.	SE Ext. trig.	EN Telephone	SE Telefon	EN Bed shaker	SE Sängvibrator
DE Trigger	NO Ekst. utl.	DE Telefon	NO Telefon	DE Vibrationskissen	NO Sengevibrator
ES Activ. ext.	DK Ekst. udl.	ES Teléfono	DK Telefon	ES Vibrador de cama	DK Sengevibrator
FR Déclench. ext.	FI Ulk. lauk.	FR Téléphone	FI Puhelin	FR Vibreux de matelas	FI Tärstin
NL Ext. trig.	PL Wyzw. zewn.	NL Telefoon	PL Telefon	NL Triltschijf	PL Wibrator łóżkowy
IT Trig.est.	CZ Ext. sig.	IT Telefono	CZ Telefon	IT Bed shaker	CZ Vibration doplněk
PT Gat. ext.	HU Külső akt.	PT Telefone	HU Telefon	PT Vibração cama	HU Ágyrezgető
GR Σκανό.	SK Ext. spúš.	GR Τηλέφωνο	SK Telefón	GR Δόνηση κρεβατιού	SK Vibracné zariadenie

EN	Technical information	25
DE	Technische Information	26
ES	Información técnica	27
FR	Informations techniques	28

NL	Technische informatie	29
IT	Informazioni tecniche	30
PT	Informação técnica	31
GR	Τεχνικές πληροφορίες	32

SE	Teknisk information	33
NO	Teknisk informasjon	34
DK	Tekniske specifikationer	35
FI	Tekniset tiedot	36

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Technical Information

Mains power	7.5 VDC / 1.5 A
Backup battery power	4 x 1.2 V AAA NiMH rechargeable batteries (Not supplied with Classic) Operating time: > 24 h Charging time: 10 h from depleted to fully charged
Dimensions and weight	121 mm (4.8") x 108 mm (4.3") x 92 mm (3.6"), 390 g (13.7 oz) incl. batteries
Alarm signal	100 dB(A) @ 10 cm, 4", 94 dB(A) @ 30 cm, 12", multi-frequency 950-3000 Hz Alarm timeout: 20 min 4 high-intensity pulsating LEDs (Not available on the Classic model)
Flash signal	Power: 2.0 – 4.0 VDC.
Bed shaker output	Decreasing from 9 to 2 min in 2 min steps. Timeout: 20 min
Snooze time intervals	Twisted Nematic (TN) LCD. Activated by the snooze button. Backlight intensity adjustable in 4 steps. Step 5 = Permanent backlight.
LCD display	Electro luminescent. Timeout: 1 h. (Not available on the Classic model)
Night light	Telephone input, RJ-11, analogue, 26-120 VRMS, 15-100 Hz
Inputs	Ext. trig. input, 3.5 mm tele (Inputs not available on the Classic model)
Buttons and controls	Separate push-rotate dials for time and alarm, snooze button, back-light intensity switch, 12/24-h mode switch, alarm on/off switch and flash/sound select switch (Not available on the Classic model)
Radio frequency	315 MHz, 433.92 MHz or 868.3 MHz depending on region (Visit model only)
Radio coverage	50 – 250 m depending on radio frequency (Visit model only). Coverage is reduced by walls, large objects and other radio transmitters.

Additional information

Operation, storage and transport conditions	Operating temperature: 0°–35° C, 32°–95° F. Transport and storage temperature: -10°–50° C. Relative humidity: 15% - 90% (non condensing).
Maintenance and cleaning	Maintenance free. Clean with a dry cloth.
Safety instructions	This product is not intended to be refurbished for re-use. To disconnect the power fully, the AC/DC adapter plug must be disconnected from the mains socket. The AC/DC adapter plug must not be blocked and must be easy to access for the intended use. The marking information is located at the bottom of apparatus. Please contact authorized service personnel for changing batteries. Replace batteries with the same battery type, Rating: 1.2V 600mAh NiMH.
Power supply	Use only the specified rechargeable batteries, other batteries may damage the device.
Warranty and service	If your Alarm clock appears to be damaged or doesn't function properly, contact your retailer for information about warranty service.

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Additional regulatory information

- a) This equipment complies with Part 68 of the FCC rules and the requirements adopted by the ACTA. On the **bottom** of this equipment is a label that contains, among other information, a product identifier in the format **US:AAAEQ##TXXXX**. If requested, this number must be provided to the telephone company.
- b) An applicable certification jacks Universal Service Order Codes (USOC) for the equipment is provided (i.e., **RJ11C**) in the packaging with each piece of approved terminal equipment.
- c) A plug and jack used to connect this equipment to the premises wiring and telephone network must comply with the applicable FCC Part 68 rules and requirements adopted by the ACTA. A compliant telephone cord and modular plug is provided with this product. It is designed to be connected to a compatible modular jack that is also compliant. See installation instructions for details.
- d) The REN is used to determine the number of devices that may be connected to a telephone line. Excessive RENs on a telephone line may result in the devices not ringing in response to an incoming call. In most but not all areas, the sum of RENs should not exceed five (5.0). To be certain of the number of devices that may be connected to a line, as determined by the total RENs, contact the local telephone company. [For products approved after July 23, 2001, the REN for this product is part of the product identifier that has the format **US:AAAEQ##TXXXX**. The digits represented by ## are the REN without a decimal point (e.g., 03 is a REN of 0.3). For earlier products, the REN is separately shown on the label.]
- e) If this alarm clock equipment causes harm to the telephone network, the telephone company will notify you in advance that temporary discontinuance of service may be required. But if advance notice isn't practical, the telephone company will notify the customer as soon as possible. Also, you will be advised of your right to file a complaint with the FCC if you believe it is necessary.
- f) The telephone company may make changes in its facilities, equipment, operations or procedures that could affect the operation of the equipment. If this happens the telephone company will provide advance notice in order for you to make necessary modifications to maintain uninterrupted service.
- g) Should you experience trouble with this equipment, please contact your retailer for information about warranty and service. If the equipment is causing harm to the telephone network, the telephone company may request that you disconnect the equipment until the problem is resolved.
- h) Please follow instructions for repairing if any (e.g. battery replacement section); otherwise do not alternate or repair any parts of device except specified.
- i) Connection to party line service is subject to state tariffs. Contact the state public utility commission, public service commission or corporation commission for information.
- j) **NOTICE:** If your home has specially wired alarm equipment connected to the telephone line, ensure the installation of this alarm clock does not disable your alarm equipment. If you have questions about what will disable alarm equipment, consult your telephone company or a qualified installer.

NOTICE: According to telephone company reports, AC electrical surges, typically resulting from lightning strikes, are very destructive to telephone equipment connected to AC power sources. To minimize damage from these types of surges, a surge arrester is recommended.

Applicable for Coin or Pay Phone Only

To comply with state tariffs, the telephone company must be given notification prior to connection for customer-owned coin or credit card phone. In some states, the state public utility commission, public service commission or corporation commission must give prior approval of connection.

FCC compliance statement

For ACTA:
REN: 0.18
US: 1PNRG01BBERXTE1

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

For a complete Declaration of Conformity please contact:
info@bellman.com or call +46 31 68 28 20

Statement for Industry Canada

This device contains licence-exempt transmitter(s)/receiver(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s). Operation is subject to

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This Class B digital apparatus complies with Canadian ICES-003.
IC: 6693A-BERXTE1 for Terminal Equipment

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Revision: BE1580_067MAN013

Date of issue: 2021-04-20

TM and © 2021 Bellman & Symfon AB. All rights reserved.

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Bellman & Symfon Group AB