



## ■ COMFORT PLUS™ VENDED TAMPONS

Comfort Plus™ Vended Tampons feature a petal tip cardboard applicator designed with comfort in mind.

### Key Features

- Regular absorbency cardboard applicator tampon
- Premium petal tip cardboard applicator provides easy insertion
- Concave grooves incorporated in applicator to improve grip
- Instructions printed right on the case
- Reduced case count saves space and reduces inventory investment
- Can be used in most non-proprietary competitive dispensers – wall mounted or recessed.



The petal tip cardboard applicator provides comfortable insertion.



Item #		Selling Unit
CPCR200	Comfort Plus™ Cardboard Applicator Tampon, Vending Tube - Regular Absorbency	200/cs

Not available for sale in Canada



P: 800.942.9199

F: 216.279.0019

[www.hospescobrand.com](http://www.hospescobrand.com)

L-HBG-0314-S | 11.17.23 DW





## COMFORT PLUS™ VENDED TAMPONS



**What you should know about Toxic Shock Syndrome (TSS)**  
The warning signs of TSS include: A sudden high fever (usually 102° F/38.8° C or higher), vomiting, diarrhea, a rash that looks like a sunburn, dizziness, muscle aches, or fainting or near fainting when standing up. TSS can rapidly progress from flu-like symptoms to a serious illness that can be fatal.

**If you have any of these signs and are using a tampon, remove it and contact your doctor for immediate treatment.**  
Tell your doctor that you have been using tampons and think that you may have TSS. You should seek medical treatment before resuming the use of tampons if you have had TSS warning signs in the past. To answer any questions you may have regarding TSS or tampon use, consult your doctor. The decision to use a tampon is, as it always has been, a personal decision. In order to make informed decisions about the use of tampons, you should be aware of the information in the bullet points below:

- 1 TSS is a rare but serious and sometimes fatal disease associated with tampon use. About half the cases are associated with tampon use; the remainder occur in men, children, and women who are not menstruating. TSS is treatable, but early recognition and treatment are important.
- 2 The incidence of TSS in the U.S. is estimated to be 1 to 17 cases per 100,000 menstruating girls and women per year. The risk of developing TSS is higher for teenage girls and women under 30 years of age than for older women.
- 3 You can reduce the risk of TSS by alternating your tampon use with pads. You can also avoid the risk of tampon-associated TSS by not using tampons.
- 4 Use a tampon with the minimum absorbency needed to control your menstrual flow in order to reduce the risk of getting TSS. Epidemiological studies have found that the risk of TSS may be related to tampon absorbency: the higher the absorbency, the greater the risk of TSS; the lower the absorbency, the lesser the risk.

**The correct way to use tampons**

- Use tampons only during menstruation.
- Always use the lowest absorbency tampon to meet your needs.
- Change your tampon at least every 4-8 hours.
- You can wear a tampon overnight for up to 8 hours. If you'd like to use a tampon overnight, insert a new tampon before going to bed, and replace it immediately upon waking. If you sleep longer than 8 hours, use a pad instead.
- Always remove your used tampon before inserting a new one.
- Be sure to remove the last tampon you use at the end of your period.

**Usage instructions**

- 1 After washing your hands, take the product out of the wrapper.
- 2 Get into a comfortable position. Most women either sit on the toilet with knees apart, squat slightly with knees bent, or stand with one foot on the toilet seat.
- 3 Insert the applicator. Hold the outer insertion tube by the Finger Grip design with your thumb and middle finger. With the removal string hanging down, insert the tip of the applicator into your vagina at a slight upward angle. Slide the outer insertion tube all the way into your vagina until your fingers touch your body.
- 4 Push the tampon inside. Push the inner tube with your pointer finger all the way into the outer tube, or use your other hand to push the inner tube.
- 5 Remove the inner and outer applicator tubes at the same time. The tampon should now be comfortably inside you, with the removal string hanging outside your body. When a tampon is inserted properly you shouldn't feel any discomfort. If you feel uncomfortable, the tampon may not be placed far enough inside. If this happens, remove the tampon and try again with a new one. Wash your hands again after inserting the tampon.

**Removal**

- 6 Relax your muscles. Slowly and gently pull the removal string downward at the same angle you used to insert the tampon. If you feel discomfort while removing the tampon, it has not yet fully absorbed. You may either want to wear a tampon longer next time or use a lower absorbency.

**Questions? Call 800-942-9199**  
Medical Dev.# D117431

**Disposal**

- 7 DO NOT FLUSH. Place the used wrapper, applicator, and tampon in an appropriate waste container.

### Instructions Printed on Box



ITEM#	CPCR200
DESCRIPTION	Comfort Plus™ Cardboard Applicator Tampon, Regular Absorbency, Vending Tube
CERTIFICATION	Class II Medical Device
CASE PACK	200/cs
SELLING UNIT	Case
CASE DIMENSIONS	15.81"D x 5.44"W x 7.13"H
CASE WEIGHT	3.2#
ABSORBENCY	Regular
APPLICATOR MATERIAL	Cardboard
UPC	075289065002
ITF-14	10075289065009
CASES/PALLET	210
TI / HI	21 / 10
INGREDIENTS	Rayon, Cotton, Polyethylene, Polyester, Wax, PEG Castor Oil, PEG Cocoate, Dialkyl Sulfosuccinate.